Produktbeschreibung

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Product description

SMOOTHSTAR BITE SPEZIAL GRÜN is an extremely hard and fast setting bite registration material. Delivered in cartridges **SMOOTHSTAR BITE SPEZIAL GRÜN** can be applied directly and bubble-free. The bite registration material is based on addition curing silicones.

SMOOTHSTAR BITE SPEZIAL GRÜN can be used for centric occlusions. It can also be applied as key material e.g. for taking intraoral registrations with needle points. Finally, it can be used for fixation of bite forks for use with face bows.

SMOOTHSTAR BITE SPEZIAL GRÜN is very suitable for cutting and contouring because of its extremely high final hardness.

Thanks to its thixotropicity, undesired flow of **SMOOTHSTAR BITE SPEZIAL GRÜN** into dental interstices is prevented and allows an easy removal. Therefore **SMOOTHSTAR BITE SPEZIAL GRÜN** can be applied very easily.

SMOOTHSTAR BITE SPEZIAL GRÜN meets the requirements of **ISO 4823, type B** (bite registration materials).

Indications/Intended use

- Bite registration material
- Key material e.g. for taking intraoral registrations with needle points
- Fixation of bite forks for use with face bows

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Preparing of the AUTOMIX-cartridge

Remove and dispose the closure-cap and replace it by a supplied mixing cannula. Mount the cartridge into the application gun and extrude needed material.

Note:

Store used cartridge with fixed used mixing cannula. **Do not re-use the cartridge cap.** Closing the cartridge by turning on the original cap between use could cause cross contamination of components and thus premature setting.

Application

Bite registration

The automatically mixed material is applied directly onto the occlusal areas. Within **20 seconds** after the beginning of the extruding (working or application time at 23 °C or 74 °F) allow the patient to close the mouth in the desired occlusion. The recommended time in mouth is about **1:00 minute**.

Do not contact the product with latex gloves, they may inhibit the setting. Do not touch teeth with latex gloves.

Articulator

SMOOTHSTAR BITE SPEZIAL GRÜN allows a visual control during fixing of the gypsum model in the articulator. Registrates made from **SMOOTHSTAR BITE SPEZIAL GRÜN** can be stored over months without shrinkage.

Disinfection

Disinfection with disinfection solutions specific for materials based on addition curing silicones is possible.

Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not use latex gloves because they may inhibit the polymerization of vinyl silicones. Do not touch prepared tooth with latex gloves.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

P. L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P. L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and P. L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P. L. Superior Dental Materials GmbH product.

Limitation of Liability

Except where prohibited by law, P. L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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