Produktbeschreibung

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Product description

SMOOTHSTAR MONO SYMBIO BLAU is a syringable **high viscosity base impression material** consisting of addition curing vinyl silicones. In conjunction with SMOOTHSTAR LIGHT FAST LILA it can serve as an ideal base for the two phase (one step) impression technique (two gun technique).

SMOOTHSTAR MONO SYMBIO BLAU shows good **hydrophilic** properties and is compatible for moist dental surfaces and yields a better reproduction. SMOOTHSTAR MONO SYMBIO BLAU meets the requirements of **ISO 4823, type 1** (heavy-bodied consistency).

Indications/Intended use

- Double-mix technique (1 step/2 phase) with e.g. **SMOOTHSTAR LIGHT FAST LILA**

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Preparation of the cartridge

Hold the cartridge with the sealed outlet openings facing upwards. Take the handle of the sealing cap and remove the sealing cap upwards. Avoid excessive force.

Insert the cartridge into the mixing machine (according to manufacturers instructions). Attach one of the supplied mixing tips to the cartridge. After the mixing tip is correctly seated (the central hexagonal socket at the lower end of the tip must be properly aligned with the hexagonal drift shaft), place the yellow Bayonet ring over the mixing tip and push it down completely. Turn the ring clockwise (1/4 turn) to lock the mixing tip securely in place.

Check whether the hexagonal drift shaft does engage when starting. The spiral in the mixing tip must rotate when starting.

If the used mixing machine offers different mixing speed levels, use always the lowest mixing speed.

Note:

Discard the first 3 cm of impression material extruded from newly filled cartridges.

Store used cartridge with fixed used mixing tip. It serves as a cap.

To remove the used mixing tip turn the Bayonet ring anti-clockwise and draw it off.

Impression taking

Load the impression tray with SMOOTHSTAR MONO SYMBIO BLAU (Fast Set) and apply SMOOTHSTAR LIGHT FAST LILA around the preparation. Tray loading and syringing around the preparation have to be completed at the same time.

Insert the tray not later than **1:30 minutes** from start of mixing (working time at 23°C or 74°F) into the mouth. Maintain position for at least **2:00 minutes**.

The impression should be stored under normal room conditions (max. 25 °C or 77 °F).

Casting

The impression can be poured **30 minutes** after removal from the mouth. No other time limitation does exist.

Electroplating

All impressions can be copper- or silverplated.

Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones.

Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not use with standard (condensation reaction type) silicone impression materials.
- Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.
- When using the double-mix technique (1 step/2 phase), make sure that both materials have the same setting time in mouth. The working time is determined by the material with the shorter working time.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

S&C Polymer Silicon- und Composite Spezialitäten GmbH warrants this product will be free from defects in material and manufacture. S&C Polymer Silicon- und Composite Spezialitäten GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and S&C Polymer Silicon- und Composite Spezialitäten GmbH's sole obligation shall be repair or replacement of the S&C Polymer Silicon- und Composite Spezialitäten GmbH product.

Limitation of Liability

Except where prohibited by law, S&C Polymer Silicon- und Composite Spezialitäten GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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